

Technical Data Sheet

HeiQ Viroblock washable mask RW-DWR

Multi-hi-tech comfort & protection

Reusable / washable & water repellent
With antimicrobial surface

Uses

Textile washable face masks treated with an antimicrobial technology HeiQ Viroblock NPJ03 giving the mask extra resistance against viruses and bacteria.

Covering the mouth, nose and lower part of the jaw, which is suitable for filtering solid, liquid, or liquid and solid non-oily particles (such as dust, droplets, pollen, willow home, haze etc.) suspended in air under special operations or every-day conditions.

*in the process of getting the FDA Emergency Use Authorization to be marketed in the USA.



Benefits and Characteristics

- ✓ **HeiQ Viroblock, HeiQ Pure, HeiQ Eco Dry, HeiQ Smart Temp**
- ✓ Three-layer construction
- ✓ Ultra-soft inside layer
- ✓ Surface treatment gives the material an extra antiviral and antibacterial protection
- ✓ Washable with 40°C, 30 times
- ✓ Adjustable ear strap
- ✓ Adjustable nose clip

Product Information

Product name	HeiQ Viroblock Washable Mask RW-DWR
Mask standard	Community mask / Aesthetic mask
Filter Materials	Top layer: 100% polyester, HeiQ Pure & HeiQ Eco Dry treated Filter 1: 44.5% wood-pulp, 55.5% PET, HeiQ Viroblock treated Filter 2: 100% polyester Inner layer: 100% polyester, HeiQ Smart Temp treated Ear strap: Cotton spandex
Size specification	L: adult man M adult lady S: smaller adult or bigger kids XS: smaller kids approximately 3-8 years old
Manufacturer	HeiQ Materials AG Rütistrasse 12 8952 Schlieren Switzerland Tel: +41 56 250 68 50
Shelf life	7 years
Country of origin	Vietnam

Antimicrobial treatment

Compliance	Compliant with EU BPR & US FIFRA
EPA Reg. No:	49403-38-85249
Treated article claim (EU)	This innovative Swiss Technology makes the fabric resistant to harmful microbes. Treated with a biocide to protect it from microbes and germs. Active ingredient: silver chloride.
Treated article claim (USA)	This innovative Swiss Technology protects textiles from microbes and germs. Antimicrobial properties built in to protect the product. Product does not protect users or others against pathogens. Always clean this product thoroughly after each use.

Usage instruction

1. After washing your hands open the mask with both hands
2. Hang the ear straps on both sides behind the ears.
3. Pull up and down the mask to fit tightly under your jaw and close the nose clip. The mask should completely cover the nose, mouth and chin.



Washing instructions

The mask is washable for 30 times at 40°C (104°F)



- Gentle wash at 40°C (104°F)
- Do not bleach
- Do not tumble dry
- Do not dry clean

Recycling

This product can be recycled as other textile products.

ATTENTION:

HeiQ Materials AG can make no warranties or representations, either expressed or implied, that these products will fully protect the user from exposure to blood or bodily fluids or risk of contracting infectious diseases. Evaluate the anticipated exposure to pathogens and contaminants and select the appropriate mask for the intended protection.

This information corresponds to the present state of our knowledge and is intended as a general description of our products and their possible applications. HeiQ Materials AG and its subsidiaries ("HeiQ") makes no warranties, express or implied, as to the information's accuracy, adequacy, sufficiency or freedom from defect and assumes no liability in connection with any use of this information. Any user of this product is responsible for determining the suitability of HeiQ's products, whether used singly or in combination with other substances, for its particular application. * Nothing included in this information waives any of HeiQ's General Terms and Conditions of Sale, which control unless expressly agreed otherwise in writing. Any existing intellectual/industrial property rights must be observed. Due to possible changes in our products and applicable national and international regulations and laws, the status of our products could change. Material Safety Data Sheets providing safety precautions, that should be observed when handling or storing HeiQ products, are available upon request and are provided in compliance with applicable law. You should obtain and review the applicable Material Safety Data Sheet information before handling any of these products. The purchaser is responsible for complying with all applicable governmental laws, regulations and standards relating to the handling, use, storage, distribution, sales, disposal of, and exposure to these products. For additional information, please contact HeiQ. * For sales to customers located within the United States and Canada the following additionally applies: NO EXPRESS OR IMPLIED WARRANTY IS MADE OF THE MERCHANTABILITY, SUITABILITY, FITNESS FOR A PARTICULAR PURPOSE OR OTHERWISE OF ANY PRODUCT OR SERVICE.

HeiQ Viroblock Washable Mask RW-DWR – ENG

Rev B: 12 June 2020



HeiQ Materials AG

Ruetistrasse 12 ▪ 8952 Schlieren (Zurich) ▪ Switzerland

Phone +41 56 250 68 50 ▪ Fax +41 56 250 68 51 ▪ info@heiq.com ▪ www.heiq.com